

Poly Medicure Limited

232- B, 3rd Floor, Okhla Industrial Estate, Phase - 3, New Delhi - 110020
 Tele: 011-26321889/26321893/26321899/26321838 Fax: 011-26321894/26321839
 Web: www.polymedicure.com E-mail: marketing@polymedicure.com
 CIN NO. L40300DL1995PLC066923

DDDC
Civild hospital
Jagadhari

Commissioning Report.

Commissioning check list:

The commissioning (putting into the service) shall be performed and documented before the machine is handed over user according to the specified check list with reference to instructions for use.

REF (Article NO.)	Dial-Ezee
SN (serial Number)	XT-210621
Date of Purchase	—
Date of Installation	16/Nov/2021
Responsible Person (User)	Miss Ridhima (Centre Manager)
Operating hours (h)	20 hrs.
Supplied by	Polymedicure Ltd.
Company	Poly Medicure Limited

	Visual inspection:	Done
A		
1	Machine: Clean /complete, no damages /moisture influences or loose assemblies; no moveable parts touching tubing's or wires; casters are moveable.	✓
2	Check tight seat and damages of mains supply (Main's cord and connectors)	✓
3	Protect Earth Resistance; < 0.3 (Ω) (Note Highest value).	✓
4	Machine Inc. Mains cord, Move the mains cord during the check, thus possible loose connections can be detected;	✓
5	check earthing on the earthing point.	✓
B	Install Machine	
1	If present remove the Transport locks (cable Ties) Connect water inlet to the machine, Connect Dialysate Connectors and fasten with tubing	✓
2	Assembly Dialyser Holder	✓
C	Function Inspection	
1	Pay attention to the filling of the machine to prevent dry run of the heater	✓
2	Switch on Machine, Fill and rinse.	✓
3	Flow chambers, Go to heating Menu increase the value, Then rinse in disinfection	✓

Complain No:-
 Mr. Prabhudyal - 9999094376
 Rajull (Er.) - 9625216183

Poly Medicure Limited

232- B, 3rd Floor, Okhla Industrial Estate, Phase - 3, New Delhi - 110020
 Tele: 011-26321889/26321893/26321838 Fax: 011-26321894/26321839
 Web: www.poly medicure.com E-mail: marketing@polymedicure.com
 CIN NO. L40300DL1995PLC066923

4	Self-test Passed (and 15 minutes with therapy with UF function check	✓
5	Ultrafiltration Comparison measurement 15 Minutes with UF rate with 500ML/h	✓
6	Temperature: compare measurement 37°	36.5°L ✓
7	Conductivity: Compare measurement 14.3ms/cm	14.0ms/cm ✓
8	Check Leakage current of the Machine: Note (All water connections and data lines must be connected during the check of the leakage current ≤ 0.5 (mA) During heat-up phase(change mains polarity and note highest value)	✓
9	Safety Air detector check (SAD), alarm function audio /visual	✓


Note: The commissioning was performed, and the machine was handed over to the Responsible person.

1. Customer signature: _____

2. Customer Name : _____

3. Date : _____

4. Signature : _____

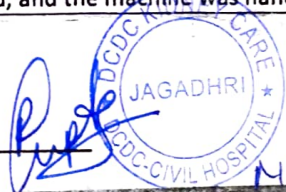

Miss Ridhima
With seal

17/Nov/2021

Prafull

Name Service Engineer: _____

PRAFULL

1. Customer signature: